G-CLOR ipoclorito di sodio N5

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G-CLOR sodium hypochlorite N5

MEDICAL DEVICE

5% Sodium hypochlorite-based aqueous solution in active chlorine ready for use to wash and clean root canals during boring.

1. **Trade name:**
G-CLOR sodium hypochlorite N5
2. **Classification**
Class IIA medical device.
3. **Composition**
Sodium hypochlorite 5% titrated by weight with active chlorine.
4. **Description**
Root canal washing solution. To be used after mechanical boring and before the final sealing with filling cement. Excellent solvent of organic residues.
5. **Indications and applications**
Suitable for irrigation of root canals before and during their treatment, in order to free them from necrotic material and residues of any kind.
6. **Instructions for use**
 - Proceed to irrigate the canal with G-CLOR using a syringe with truncated and curved-tip needle.
 - Insert the needle at the entrance of the canal, being careful not to go too deep into the apex.
 - By applying gentle pressure on the syringe plunger, sway the needle slightly back and forth, so as to facilitate the inflow and outflow of the solution in the canal. To use a period of time not exceeding 1 minute.
 - For every wash using G-CLOR, perform a second wash with hydrogen peroxide in 12 volumes, following the same procedure. The hydrogen peroxide will create an effervescence that will make it easier to clean the canal.
 - To remove inorganic debris that has formed when using the equipment, we recommend alternating G-CLOR with EDTA.
 - Finally, following the same procedure, perform a wash with sodium chloride 0.9% (sterile saline) and dry thoroughly with sterile absorbent paper points.

The canal is now ready for the subsequent treatment. This procedure is required to prepare the canal for the final sealing treatments with filling cement.

7. **Precautions, Safety and Environmental impact**
For the washes, we recommend that you use specific needles for root canal washing which, because they are blunt, avoid the risk of accidentally injecting the product. Remember also to use the appropriate dam to prevent the product from stagnating on the mucosa and thus possibly causing irritation. Wear suitable protective clothing, gloves and glasses. Do not mix the product with acids, since it would release toxic gases.
8. **Warnings and Safety tips**
EXTERNAL USE ONLY - NOT FOR INJECTION - DO NOT INGEST

Hazard statements:

- H400 - Very toxic to aquatic organisms.
- H411 - Toxic to aquatic organisms with long-lasting effects.
- H290 - May be corrosive to metals.
- H314 - Causes severe skin burns and serious eye injury.

Precautionary statements:

- P101: In case of consulting a physician, have the container or product label available.
- P102: Keep out of reach of children.
- P264: Wash thoroughly after use.
- P280: Wear protective gloves/protective clothing/eye/face protection.
- P301+P330+P331: IF INGESTION: rinse mouth. DO NOT induce vomiting.
- P303+P361+P353: IF ON SKIN (or hair): remove all contaminated clothing immediately. Rinse skin or take a shower.
- P305+P351+P338: IF IN EYE CONTACT: rinse thoroughly for several minutes. Remove any contact lenses if it is convenient to do so. Continue rinsing.
- P501: Dispose of the product/recipient in the licensed recycling collection system at your municipality.

More or less yellowish color variations from batch to batch do not affect the quality of the product. Do not use the product beyond the expiration date.

Additional information:

- EUH 031 in contact with acids releases toxic gas;
- EUH206: Caution. Do not use in combination with other products. Hazardous gases (chlorine) may be released.

9. **Quality controls**
The Company applies a full-scale quality system, in compliance with UNI EN ISO 9001:2015 - UNI CEI EN ISO 13485:2021.
10. **Shelf life**
24 months for the product in unopened, properly preserved package.
11. **Storage instructions**
G-CLOR is light-sensitive and should therefore be kept away from light and heat sources.
12. **Nature and capacity of containers**
Glass bottle (250 ml). G-CLOR is a class IIA medical device [Directive No. 17/EEC updating Directive No. 2007/47/EEC, transposed into Law in Italy by Legislative Decree No. 46/97, updated by Legislative Decree No. 37/2010].

This publication is reserved exclusively for qualified health professionals. The document may undergo revisions to introduce improvements, regulatory and legislative developments, or changes of other nature.

13. Symbols

REF
Product code

LOT
Lot number

Use by

Keep away from sunlight

Irritating

Not-sterile

IFU

Manufacturer

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it
web: www.dental-world.it

Simit Dental Srl
Via Carlo Pisacane 5/A - 46100 Mantova (MN)
Tel. +39 0376 267811 - Fax. +39 0376 381261

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